

INSTRUCTIONS

ImPACT AT-HOME TESTING

1

Find the email that contains your unique testing link. (Note: it may have gone to your spam folder).

- a. If you cannot find the email, reach out to the organization who assigned you the test to re-issue the link.

2

Before continuing, make sure of the following:

- a. Your computer is plugged into an electrical outlet.
- b. Pop-up blockers are turned off.
- c. Test environment is quiet and distraction-free.
- d. You are well-rested and not under the influence of alcohol or drugs.
- e. Cell phones and other devices are turned off or away from the testing area.
- f. You're sitting at a table or desk with a flat hard surface.
- g. All other browser tabs and programs are closed.
- h. An external mouse, a trackpad or an iPad must be used throughout the exam.
- i. You have your ImPACT Passport ID ready (If you've taken an ImPACT baseline or post-injury test before).

3

When you're ready to start the test, click the unique link in the email and click "Validate".

- a. Having trouble launching the test? [Get troubleshooting tips.](#)
- b. The test must be completed in one sitting, with no pauses or breaks.

4

After you complete the test, you'll receive your [ImPACT Passport ID](#) via email (Only a trained healthcare provider can access your test results.)

- a. [Download the ImPACT Passport app](#) to safely store it so you can grant access to your test results to a trained healthcare provider when needed.

If you have any questions along the way, [visit our Help Center.](#)



ConcussionManagement.com